

A wide range of Seni absorbent products will allow you to say goodbye to the feeling of uncertainty and embarrassment associated with urinary incontinence and will provide excellent protection. **Choose the right absorbent product according to the degree of urinary incontinence and your physical condition.**

	MOBILE 	MOVES WITH HELP (walking stick, walker, on the wheelchair) 	IS BEDRIDDEN
LIGHT 	Seni Lady Slim Micro, Slim Micro Plus, Slim Mini, Slim Mini Plus, Slim Normal, Slim Extra 	Seni Man Normal, Extra 	Seni Lady Plus Seni Man Super San Seni Prima
MODERATE 	Seni Lady Extra Plus, Super, Plus Seni Active Normal Seni Active 	Seni Man Super Seni Air Classic 	Seni Air Classic Seni Optima Super
HEAVY 	San Seni Uni, Maxi, Plus Seni Active Plus Seni Optima Super Seni Optima Plus Seni Optima Trio 	Seni Active Plus Seni Optima Super Seni Optima Plus Seni Optima Trio Super Seni Super Seni Plus Super Seni Trio 	Seni Optima Super Seni Optima Plus Seni Optima Trio Super Seni Super Seni Plus Super Seni Trio

Additional protection of bed, wheelchair, armchair when changing absorbent products.

Seni Soft underpads



Proper skin hygiene is essential in the prevention of chafes and bedsores. The **Seni Care** series of skin care products has been created especially for people whose skin is sensitive and requires special protection.



How to choose the right absorbent product for urinary incontinence?

1 Determine the level of urinary incontinence

Based on several days of observations of the occurrence of urinary incontinence, determine the frequency and approximate amount of uncontrolled leakage of urine. Mark what degree of urinary incontinence you think it is:

LIGHT it happens to pass a small amount of urine between visits to the toilet <input type="checkbox"/>	MODERATE urinary incontinence is frequent and quite abundant <input type="checkbox"/>	HEAVY urinary incontinence is frequent and abundant, a complete lack of control over urine loss <input type="checkbox"/>	Does fecal incontinence occur additionally? <input type="checkbox"/> YES <input type="checkbox"/> NO
---	--	---	---

2 Determine the physical condition of the person for whom absorbent product is selected

MOBILE <input type="checkbox"/> leaves home on his/her own, leads an active lifestyle	MOVES WITH HELP <input type="checkbox"/> with the support of a walking stick or a walking frame, moves on a wheelchair	BEDRIDDEN <input type="checkbox"/>
---	--	--

3 Choose the right product size

Using a tailor's tape, measure the circumference of the hips and waist. Then compare these values with the sizes of Seni absorbent products listed in the table.



An example of how to choose the right size in the case of Super Seni all-in-one adult diapers.

If both dimensions: waist and hips circumference fall within one of the ranges given in the table - choose this size.



size	range in cm	range in inches
XS	40-60	16-24
S	55-80	22-31
M	75-110	30-43
L	100-150	39-59
XL	130-170	51-67
XXL	160-210	63-83

Proper size of absorbent product is very important. **Too big** product can cause leakages. **Too tight** product can cause irritations.




tzmo-global.com
www.tzmo.in

Contact us:
Phone: 080 2674 2644
WhatsApp: +91 725 900 13 20
Email: customercare.in@tzmo-global.com
Find more information on:
www.seni-india.com



Photos in the material are from photo stocks. People presented on the pictures are the models.

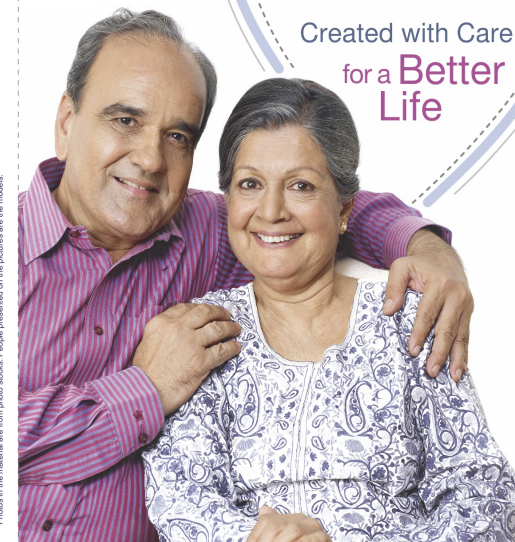

Super Seni
Breathable Adult Diapers
for people with urinary incontinence





seni®

Contrary to popular belief, it is not just aging that causes urinary incontinence. This ailment is a condition accompanying many diseases and it may be either a symptom or an effect of them. However, there is a simple and convenient way to deal with urine loss.

If you care for a person with urinary incontinence – make him or her comfortable on a daily basis.

If the problem of urinary incontinence concerns you – take care of your comfort and well-being.

Super Seni **breathable** adult diapers, thanks to their unique features, provide optimal protection in urinary incontinence and guarantee a feeling of dryness and comfort every day.


Urinary incontinence can occur at any age and regardless of gender



Try Super Seni absorbent products

classic breathable adult diapers which provide safe and effective protection in heavy and very heavy urinary incontinence.

IMPORTANT
before use,
optimize the efficiency
of the product
by activating it

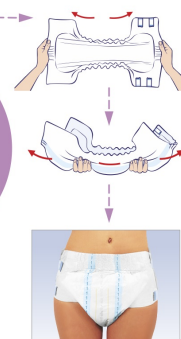

- ① **Flexi 360°** – comfort and fit
- elastic waistband at the front and back
- elastic combi-tape fasteners with the possibility of multiple closing and opening without the risk of tearing the outer layer of the adult diapers.
- ② **Air** – the outer layer of Super Seni adult diapers allows the skin to breathe easily, reducing the risk of skin problems.

- ③ **Odour Stop** – special granules (superabsorbent) in the absorbent core prevent the formation of unpleasant odour.
- ④ **Latex Free** – without latex elements - minimizing the risk of allergic reactions and allergies.
- ⑤ **EDS** – Extra Dry System quickly distributes the moisture inside the absorbent layer.

- ⑥ **Inner hydrophobic side gathers** – prevent leakage from the side of the absorbent product.
- ⑦ **Wetness indicator** – precise information about the filling level of the diaper and the necessity to change it.
- ⑧ **Fit & Dry** – double absorbent layer in the central area where dryness is the most essential.

Super Seni breathable adult diapers guarantee:

feeling of dryness

Super Seni adult diapers are available in 3 absorbency levels, so you can choose the option that suits your needs:

- **Super Seni** – for heavy incontinence,
- **Super Seni Plus** – for heavy and very heavy incontinence,
- **Super Seni Trio** – increased absorbency for very heavy incontinence.

optimal fit

- each of the absorbency levels is available in 4 or 5 sizes so the risk of side leakages (too large product) or abrasions (too small product) is minimized,
- the system of flexible fasteners makes the product work together with the body.

safety

- breathable adult diapers, like other products under the Seni brand, have successfully passed independent dermatological tests, which means that they are safe for the skin.



6
XS S M L XL



7
XS S M L XL XXL



8
S M L XL